



### Radica Diagnostic Procedures

Radica Resin can be used as an esthetic, diagnostic display, as well as a provisional. Either way you choose to use the material, it must be processed identically. You must cure all restorations, diagnostic or provisional, before delivery to your accounts.



Working model.



Make 1.0 mm depth cuts.



Uniform 1.0 mm reduction.



Wax to full contour.




Make a silicon matrix.




Model Release.



Enamel.

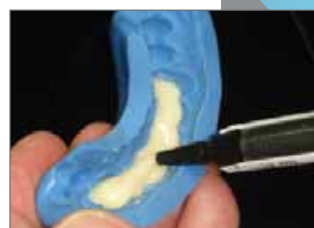

Dentin.

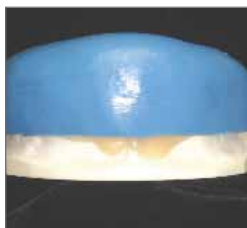

Matrix in place.



Trimming.



Sealer.



Curing.



Completed Diagnostic/Provisional.

### TABLE 1. CURE TIMES FOR DENTSPLY CURING UNITS

	<b>Enterra™ Curing Unit</b>	<b>Triad® 2000 Curing Unit</b>	<b>Eclipse® Processing Unit</b>
Initial cure	5 min	10 min	3 min
Pontic cure	1.5 min	2 min	1/2 min
VLC Sealer	2 min	2 min	1 min

Note: Do not use early versions of the Triad Curing Unit.



DENTSPLY International Inc.  
570 West College Ave.  
York, PA 17405-0872,  
717-699-4190  
www.ceramco.com  
www.dentsply.com

EC REP DeguDent GmbH  
Postfach 1364  
63403 Hanau  
GERMANY  
+49/6181/59-50

Distributed by:  
DENTSPLY Canada  
161 Vinyl Court  
Woodbridge, Ontario  
L4L 4A3, 905-851-6060

CE R only  
0086  
REF 908204  
Rev. C 07/07